



www.archive.org  
415.561.6767  
415.840-0391 e-fax

Internet Archive  
300 Funston Avenue  
San Francisco, CA 94118

## AFFIDAVIT OF CHRISTOPHER BUTLER

1. I am the Office Manager at the Internet Archive, located in San Francisco, California. I make this declaration of my own personal knowledge.

2. The Internet Archive is a website that provides access to a digital library of Internet sites and other cultural artifacts in digital form. Like a paper library, we provide free access to researchers, historians, scholars, and the general public. The Internet Archive has partnered with and receives support from various institutions, including the Library of Congress.

3. The Internet Archive has created a service known as the Wayback Machine. The Wayback Machine makes it possible to surf more than 450 billion pages stored in the Internet Archive's web archive. Visitors to the Wayback Machine can search archives by URL (i.e., a website address). If archived records for a URL are available, the visitor will be presented with a list of available dates. The visitor may select one of those dates, and then begin surfing on an archived version of the Web. The links on the archived files, when served by the Wayback Machine, point to other archived files (whether HTML pages or images). If a visitor clicks on a link on an archived page, the Wayback Machine will serve the archived file with the closest available date to the page upon which the link appeared and was clicked.

4. The archived data made viewable and browseable by the Wayback Machine is compiled using software programs known as crawlers, which surf the Web and automatically store copies of web files, preserving these files as they exist at the point of time of capture.

5. The Internet Archive assigns a URL on its site to the archived files in the format `http://web.archive.org/web/[Year in yyyy][Month in mm][Day in dd][Time code in hh:mm:ss]/[Archived URL]`. Thus, the Internet Archive URL `http://web.archive.org/web/19970126045828/http://www.archive.org/` would be the URL for the record of the Internet Archive home page HTML file (`http://www.archive.org/`) archived on January 26, 1997 at 4:58 a.m. and 28 seconds (1997/01/26 at 04:58:28). A web browser may be set such that a printout from it will display the URL of a web page in the printout's footer. The date assigned by the Internet Archive applies to the HTML file but not to image files linked therein. Thus images that appear on a page may not have been archived on the same date as the HTML file. Likewise, if a website is designed with "frames," the date assigned by the Internet Archive applies to the frameset as a whole, and not the individual pages within each frame.

6. Attached hereto as Exhibit A are true and accurate copies of printouts of the Internet Archive's records of the HTML files or PDF files for the URLs and the dates specified in the footer of the printout (HTML) or attached coversheet (PDF).

7. I declare under penalty of perjury that the foregoing is true and correct.

DATE: \_\_\_\_\_

9/6/16

Christopher Butler

CALIFORNIA JURAT

---

See Attached Document.

A notary public or other officer completing this certificate verifies only the identity of the individual who signed the document to which this certificate is attached, and not the truthfulness, accuracy, or validity of that document.

State of California  
County of San Francisco

Subscribed and sworn to (or affirmed) before me on this

6 day of SEPTEMBER, 2016, by



Christopher Butler,

proved to me on the basis of satisfactory evidence to be the person who appeared before me.

Signature: \_\_\_\_\_



# Exhibit A

[ABOUT US](#)  
[CONTACT US](#)  
[CAREERS](#)  
[SITE MAP](#)

[English \(Australia\)](#) > [Clinicians](#) > [Product Information](#) > [CPAP Masks](#) > [Ultra Mirage™ Full Face Mask](#)


## Ultra Mirage™ Full Face Mask

[Brochure](#)

[Ultra Mirage™ Full Face Mask](#) | [Components](#) | [Fitting](#) | [Cleaning](#) | [Assembly](#)

### The new way to get on top of leak & therapy management

Mouth breathing is a common problem for CPAP and bilevel therapy users. In fact, around 40% of CPAP users and almost all bilevel users experience mouth breathing and [mouth leak](#)<sup>1,2,3</sup>. The result is loss of therapy effectiveness, loss of comfort, and disrupted sleep<sup>4</sup> – all of which can lead to loss of compliance, and ultimately loss of patients on therapy. The Ultra Mirage Full Face Mask – the best of the Ultra Mirage technology, now in a full face mask – is designed to help you stop the loss.



### Features & benefits

- Popular Ultra-style forehead support enhances stability by minimizing movement of the mask relative to the face; and offers four position options to optimize seal and patient comfort.
- Soft, dual-wall cushion reduces pressure on the nasal bridge and optimises seal comfort – even when the patient moves during sleep.
- Flexible forehead pads conform to the contour of the patient's forehead, further enhancing stability and comfort.
- Top and bottom headgear clips allow quick and easy fitting, as well as quick removal of the mask in an emergency.
- Set & Forget headgear means that once the headgear is set up, it doesn't have to be readjusted each time the mask is used.
- Minimal dead space, superior airflow characteristics, and built-in air vents reduce CO<sub>2</sub> rebreathing. Even at low pressures, the mask provides excellent CO<sub>2</sub> washout.

### Now you CAN stop the loss...

Losing air is losing therapy. With ResMed's new Ultra Mirage Full Face Mask, you can beat the [mouth leak cycle](#) and **stop the loss!**

### Product Codes

16981	Fitting template
60660	Small, standard — Australia
60661	Small, shallow — Australia
60662	Medium, standard — Australia
60663	Medium, shallow — Australia
60664	Large, standard — Australia
60665	Large, shallow — Australia

This page last updated: 4 January 2005

Contact Us : [Legal Disclaimer](#) : [Intellectual Property](#) : [Privacy Notice](#) : [Site Map](#)

©2000-2016 ResMed. All rights reserved.

<http://web.archive.org/web/20060823004503/http://resmed.com.au/Shared/StaticFiles/ultra-mirage-ffm/ultra-mirage-ffm-brochure-clinician-english.pdf>






## Ultra Mirage™ Full Face Mask

The ultimate solution for mouth leak

Mouth breathing and leak are common problems for CPAP and bilevel therapy users. These problems result in ineffective therapy, discomfort and disrupted sleep—all of which can cause patients to be noncompliant or to abandon therapy altogether.

ResMed's Ultra Mirage™ Full Face Mask covers both the nose and mouth so patients receive effective therapy even in the presence of mouth breathing and leak.

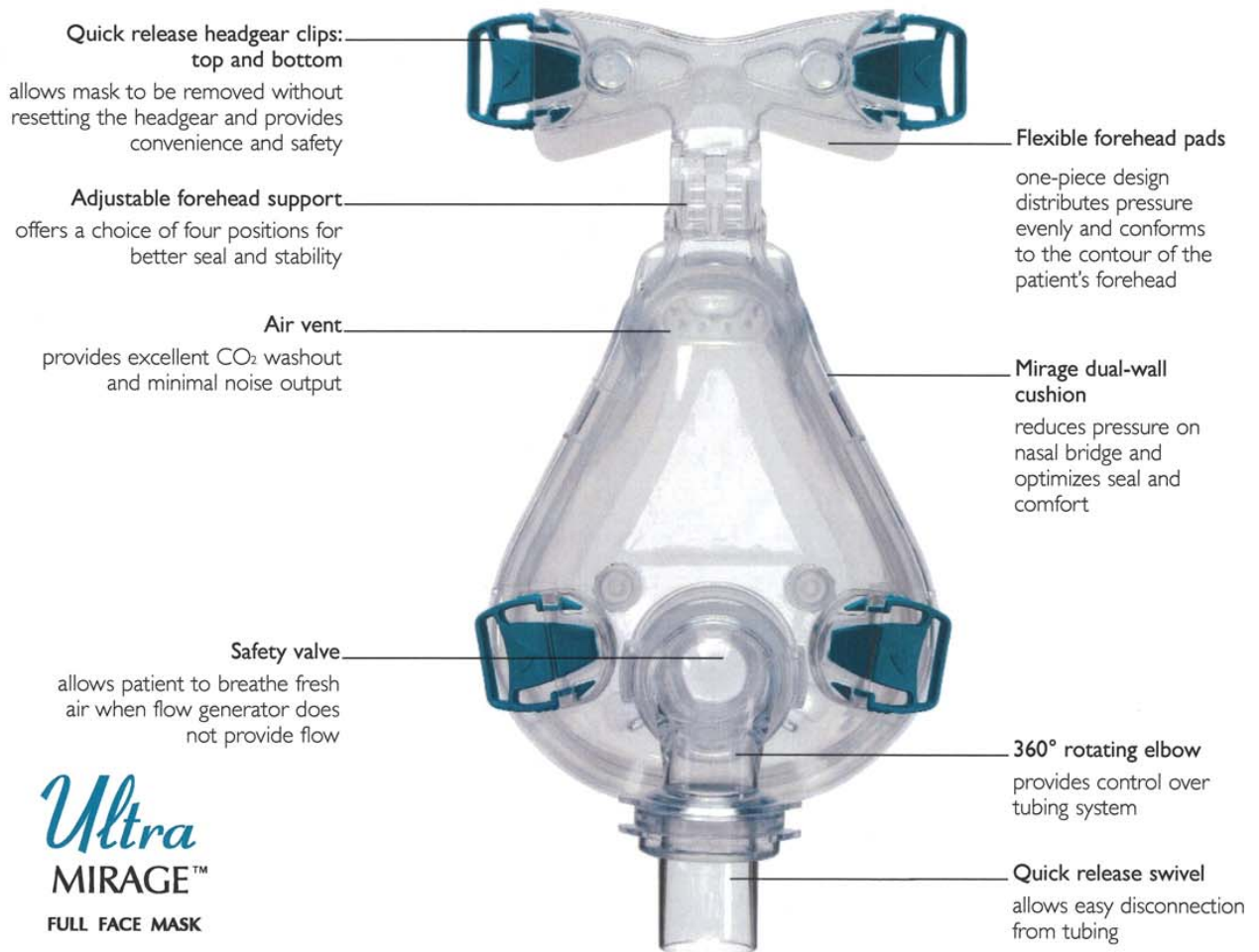

## Technology highlight—adjustable forehead support and pads

The adjustable forehead support takes seal performance to a new level.

- Enhanced stability is achieved by minimizing movement of the mask relative to the face. Seal is maintained even when the patient moves during sleep.
- Four adjustable positions optimize seal and comfort.
- The flexible forehead pads conform to the contour of the patient's forehead.
- The pads distribute pressure evenly over a larger surface area, reducing pressure on the patient's face.



### Patient benefits

- **Trusted comfort with proven technology**  
Mirage cushion technology reduces pressure on the nose and optimizes seal and comfort.
- **Superior seal through enhanced stability**  
Adjustable forehead support and pads help maintain seal even with patient movement.
- **User convenience**  
Quick release headgear clips enable quick and easy mask fitting and removal. Once set-up, the headgear doesn't have to be readjusted.

### Professional benefits

- **Confidence in effective therapy delivery**
  - As the mask covers both the nose and mouth, it provides effective therapy in the presence of mouth breathing, and
  - Minimizes the effects of mouth breathing to help optimize patient compliance.

- **Peace of mind**
  - Minimal dead space, superior airflow characteristics, and the built-in air vent minimize CO<sub>2</sub> rebreathing.
  - Quick release headgear clips allow easy removal.
  - Built-in safety valve allows patient to breathe fresh air when the flow generator provides no airflow.
- **Efficient mask size selection**
  - 3-D fitting templates help achieve first-time fit.
  - Templates may also help reduce mask fitting time and minimize call backs and returns.

## Ultra Mirage Full Face Mask System Product codes

### North & Latin America

<b>Small Standard</b>	60600
<b>Small Shallow</b>	60601
<b>Medium Standard</b>	60602
<b>Medium Shallow</b>	60603
<b>Large Standard</b>	60604
<b>Large Shallow</b>	60605

### Europe (except UK) and Asia Pacific

<b>Small Standard</b>	60610
<b>Small Shallow</b>	60611
<b>Medium Standard</b>	60612
<b>Medium Shallow</b>	60613
<b>Large Standard</b>	60614
<b>Large Shallow</b>	60615

### UK

<b>Small Standard</b>	60684
<b>Small Shallow</b>	60685
<b>Medium Standard</b>	60686
<b>Medium Shallow</b>	60687
<b>Large Standard</b>	60688
<b>Large Shallow</b>	60689

ResMed Corp Poway, CA, USA +1 858 746 2400 OR 1 800 424 0737 (toll free), ResMed Ltd Bella Vista, NSW, Australia +61 (2) 8884 1000 or 1 800 658 189 (toll free). Offices in Austria, Finland, France, Germany, Hong Kong, Japan, Malaysia, Netherlands, New Zealand, Singapore, Spain, Sweden, Switzerland, United Kingdom (see web site for details). Protected by patents: AU 710733, AU 712236, AU 724360, AU 741003, AU 744593, AU 753244, AU 766623, AU 775051, DE 29724224, CA 2261790, EP 0956069, NZ 513052, NZ 526165, US 6112746, US 6119693, US 6357441, US 6374826, US 6463931, US 6491034, US 6513526, US 6532961, US 6561190, US 6561191, US 6581602, US 6634358, US 6701927. Other patents pending. Protected by design registrations: CN 200330102703.8, CN 200330102704.2, JP 1212687, JP 1214582, US D493885. Others pending. Ultra Mirage and Mirage are trademarks of ResMed Ltd. © 2004 ResMed Ltd. 1010151/2 04 11

<http://web.archive.org/web/20060901025356/http://resmed.com.au/Shared/StaticFiles/ultra-mirage-ffm/ultra-mirage-ffm-cc-english-row.pdf>




## Ultra Mirage™ Full Face Mask


### Ultra Mirage™ Full Face Mask

(complete system with headgear and cushion)

### COMPONENTS CARD

	ROW	UK	France	Australia and New Zealand
<b>Small, Standard</b>	60610	60684	60690	60660
<b>Small, Shallow</b>	60611	60685	60691	60661
<b>Medium, Standard</b>	60612	60686	60692	60662
<b>Medium, Shallow</b>	60613	60687	60693	60663
<b>Large, Standard</b>	60614	60688	60694	60664
<b>Large, Shallow</b>	60615	60689	60695	60665

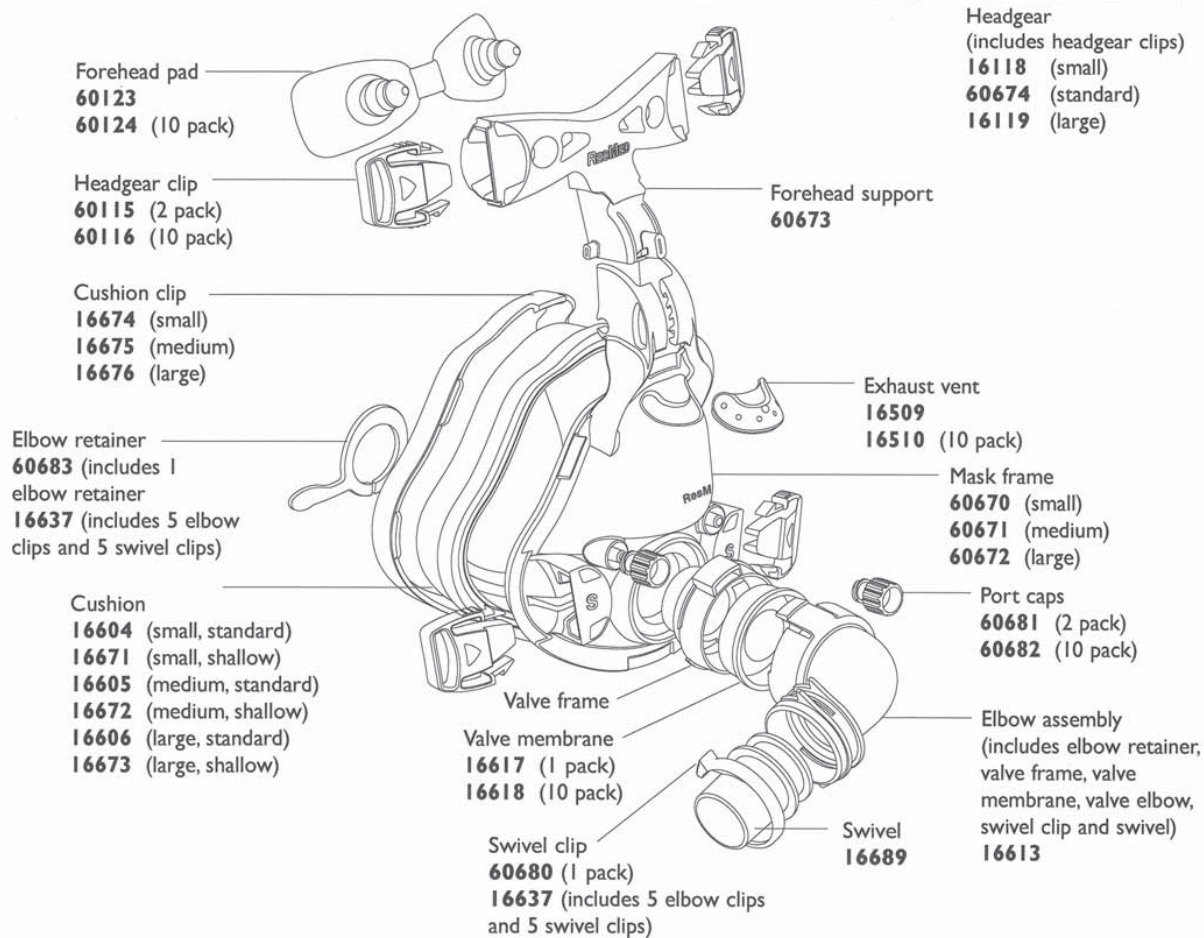

Ultra Mirage is a trademark of ResMed Ltd.   
 ©2005 ResMed Ltd. Reorder No: 1010048/4 05 12 0120